CTS - Cubital Tunnel Splint

EN ELBOW ORTHOSIS

DE ELLENBOGENORTHESE

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IT ORTESI PER GOMITO

NL ELLEBOOGORTHESE

DK ALBUEORTOSE

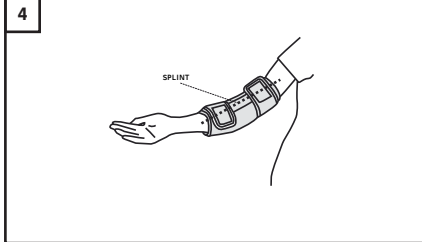
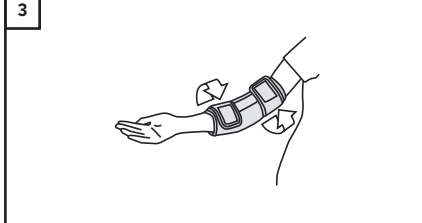
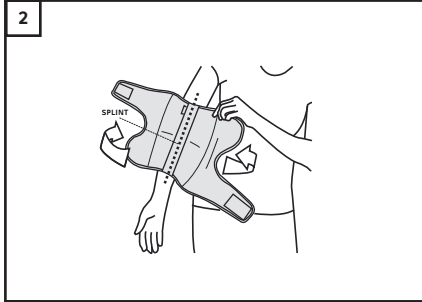
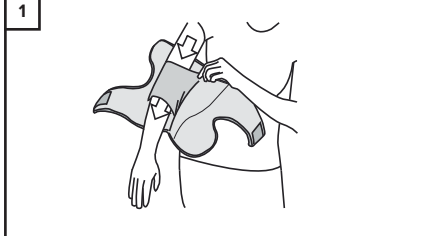
FI KYYNÄRNIVELORTOOSI

SE ARMBÅGSHORTOS

NO ORTOSE, ALBUE

PL ORTEZA ŁOKCIA

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ENGLISH

Product Description

A universal easy to fit brace that effectively restricts elbow motion. A soft padded wrap with an adjustable aluminium stay.

Usage Information (See diagram)

- Open all hook / loop fastenings.
- Position elbow in brace as shown (slide arm through Inner sleeve).
- Close hook / loop fastening.
- Re-adjust for comfort.
- Take care not to over tighten brace, re-adjust for comfort as required while ensuring brace is in correct position.
- Note: The elbow stay can be removed, bent to the desired position (as required by the injury /condition) and re-inserted. Metal stay is designed to sit on the anterior aspect of the elbow joint.

Indications for Use

A universal brace that restricts elbow flexion or extension depending on the clinical condition. Can be used for cubital tunnel syndrome or any elbow conditions where restriction / support of elbow is desired. It is recommended that individuals seek guidance from a medical practitioner regarding the most appropriate brace and wearing schedule.

Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any materials listed in the product composition.
- Perform regular skin and circulation checks.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

- Close over all hook/loop straps.
- Hand wash orthosis in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.
- Periodically check hook/loop fasteners are operating securely.

Product Family Composition

Nylon, Polyester, Polyamide, Aluminium, Elastane, Lycra.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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