Gebrauchsanweisung • Instructions for Use •
• Mode d'emploi •

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11. Entsorgung

Hausmüll

Care Instructions & Instructions for Use: Silicone Breast Prostheses

Thank you for choosing ABC Breast Care. ABC products are developed and manufactured with great care and knowledge of the needs of women who have undergone breast surgery. This instruction manual applies to ABC silicone breast prostheses.

This instruction manual DOES NOT apply to:

- Attach Breast Prostheses,
- Custom Breast Prostheses
- Textile Breast Prostheses

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Please note that not all ABC products are available in all countries.

Scope of delivery:

- Travel case for storage and travel
- Instruction manual

1. Indication for Use

- ABC breast prostheses and partial breast prostheses are intended for external use only.
- ABC breast prostheses and partial breast prostheses provide an anatomical and weight-based replacement of removed or missing breast tissue. This can be as a result of breast surgeries (such as mastectomy and/or breast conserving surgery), breast malformation, breast regression or natural breast asymmetry.
- ABC breast prostheses are intended to be used inside pocketed specialty bras. ABC breast prostheses are worn in especially designed bras with sewn-in textile pockets. Specialty bras secure the breast prosthesis in place and ensure a safe and comfortable fit.

- ABC partial breast prosthesis can be worn directly on the skin in a well-fitting bra.
- The correct breast prosthesis and appropriate specialty bra are selected and fitted by trained professionals. Shape, size, weight and color of the breast prosthesis are chosen according to the anatomical conditions and needs of the wearer.
- ABC breast prostheses and partial breast prostheses are intended for daily use and for wearing over longer periods of time.

2. Risks & Contraindications

- There are no known risks or contraindications to be expected when wearing ABC Breast prostheses and partial breast prostheses under normal conditions and in accordance with the care instructions. No safety issues or performance impairments are expected during normal use. No restrictions on use are expected.
- We recommend wearing breast prostheses and partial breast prostheses only after the wound has fully healed and radiotherapy is complete.
- If you experience redness or rashes, please discontinue using the product and contact your doctor or medical device retailer.

3. Material

- ABC silicone breast prostheses are made of polyurethane film, silicone gel based on polydimethylsiloxane, color pigments and microspheres (in lightweight prostheses).
- ABC breast prostheses are carefully controlled. Any discoloration of the breast prosthesis has no effect on the quality or wearing comfort.

4. Care Instructions

What to avoid:

- Avoid sharp objects. Be careful with sharp, pointed objects. Brooches, needles, fingernails, scissors, pet claws, and other sharp objects can cause permanent damage to your breast prostheses.
- Avoid stretching, twisting, friction or excessive use of force when handling your breast prosthesis.
- Avoid direct contact with perfume, hot surfaces or hot water.
- The breast prosthesis is resistant to sweat, moisture, water (both salt and chlorine water).

5. Cleaning Instructions

- Clean your ABC breast prosthesis daily in lukewarm water with mild soap and dry with a soft towel.
- Rinse your prosthesis with lukewarm water after wearing or swimming in salt or chlorinated water.
- Store your breast prosthesis with the cup side down in the ABC travel case.

6. Warranty

- When worn under normal conditions and in compliance with the instructions for use, we guarantee a product service life of two (2) years.
- Should your breast prosthesis become defective, please contact your medical device retailer. In the event of a production or material defect, ABC will replace the breast prosthesis should the defect happen within two (2) years from the date of purchase.
- The 2-year warranty does not apply to the ABC Swim Form and ABC Active Form

7. Incident Reporting

Only serious incidents relating to the product, which lead to a serious deterioration of the state of health or death, must be reported to the manufacturer or competent authority of the EU Member State. Serious incidents are described in Article 2(65) of Regulation (EU) 2017/745 (MDR).

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ABC Breast Care GmbH