**35305  
SELECTION  
CHILDRENS WRIST**

**35305 SELECTION  
HANDLED BARN**

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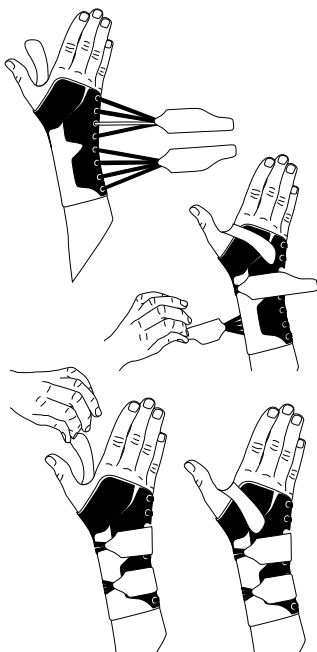
## 35305 SELECTION WRIST CHILDREN

Carefully read this instruction sheet before use!

### Intended use

To guide the hand and wrist to a functional position. To alleviate pain in inflammation and rheumatic disorders. To reduce spasticity in neurological conditions in the hand and wrist.

### Indications

Rheumatic disorders. Neurological disorders where wrist position need mild correction or support. After casting. Inflammatory conditions.

### Contraindications

No known.

### Adjustments

Fitter should make sure the stay is properly adjusted. This product should only be altered by a medical professional person.

### Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

### Material Specifications

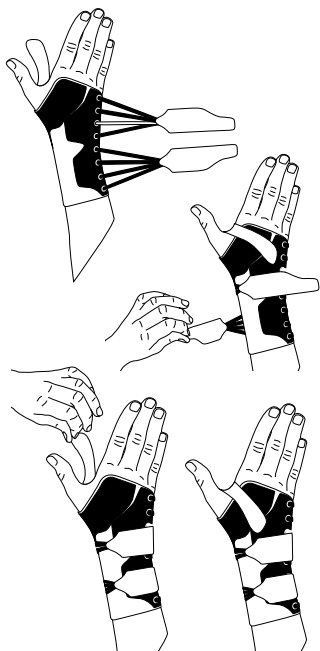
This product is made from polyester, nylon and elastan. Spoons are made from aluminum. This product is not flame retardant.

### Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F) gentle cycle. Remove stay before laundering. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

### Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

## Instructions for use

1. Check that the support is complete and without damage.
2. Open the Velcro straps and release the lacing.
3. Pull the orthosis on like a glove.
4. Secure the two straps around the wrist.
5. Secure the thumb strap.
6. Adjust for comfort.

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