**C** | **S**

**47010, 47011, 47012,  
47013 NECK COLLAR**

## 47010, 47011, 47012, 47013 HALSKRAGE

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# ENGLISH

## Indications / Intended Use

This soft neck collar is intended to provide relief to the neck for those with muscle pain or other pain relating to rheumatism.

## Contraindications

None known.

## Adjustment

This product should only be altered by a medical professional.

## Warning

Do not pull too tight, as this can cause poor circulation and/or skin irritation. If you feel any irritations or other problems, take it off immediately. Check skin on a daily basis.

## Material specifications

82% Bomull, 15% Nylon, 3% Spandex.

This product does not contain Latex.

This product is not flame retardant.

## 47010, 47011, 47012, 47013 NECK COLLAR

Carefully read this instruction sheet before use!

## Washing instructions




Close all Velcro prior to laundering. Hand wash with mild detergent. Dry flat. Press the water out from the foam. Do not twist or squeeze to remove excess water. Do not tumble dry or dry clean. This product is not flame retardant.

## Who to contact

If any problems occur please contact your orthotist or fitter.

## Time of use:

Follow your medical professionals guidelines for use and limitations. This product is recommended for single person use only.

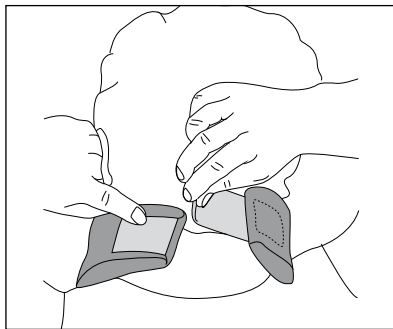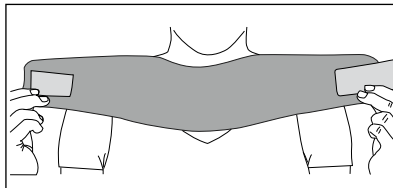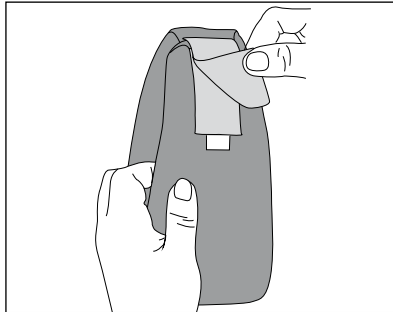## Instructions for use

1. Check that the product is complete and without damage.
2. Open the Velcro.
3. Position the collar so the closure is at the back of the neck. In some cases your medical professional may require the closure to be in the front. Please check with prescriber if uncertain.

*Model 47012 and 47013 both have an anatomical shape which should sit under your chin.*

4. Close the collar at the back of the neck.
5. Collar should fit snugly but not restrict circulation.
6. Adjust for comfort.

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Camp Scandinavia AB, Karbingatan 38