Stomacare

EasyCut *Plus*

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EN Instructions for use

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NO Bruksanvisning

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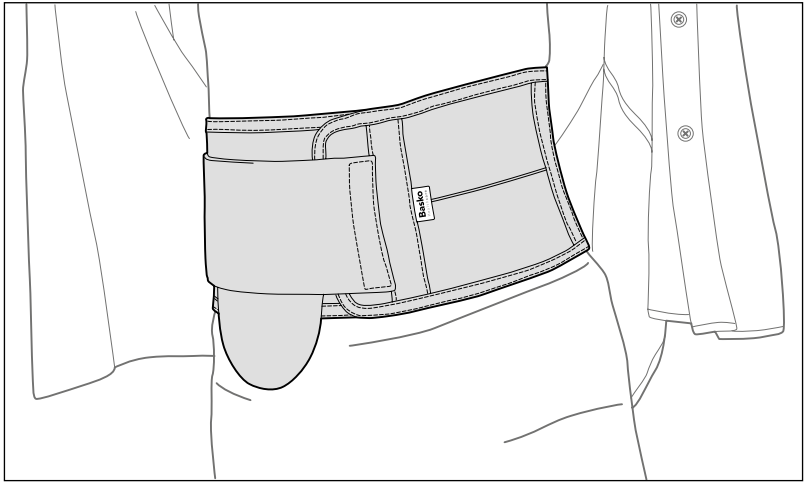
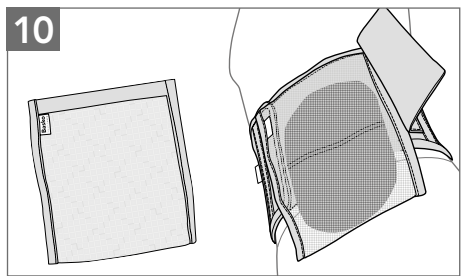
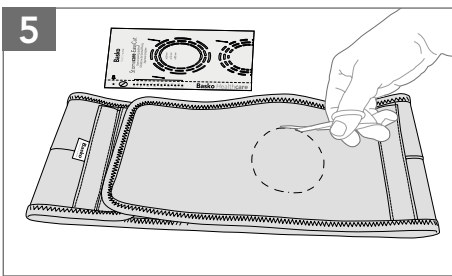
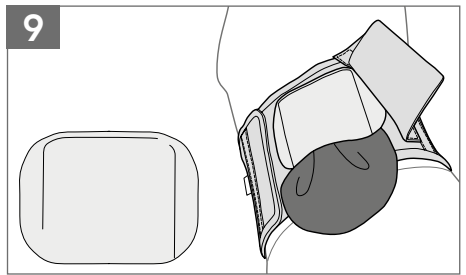
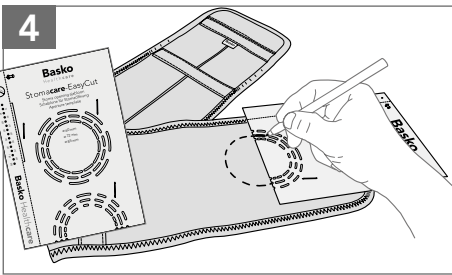
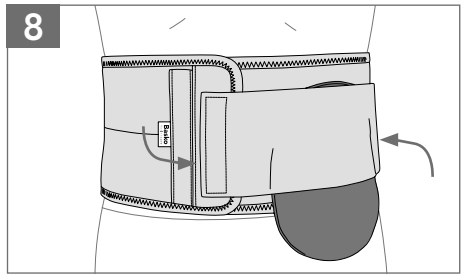
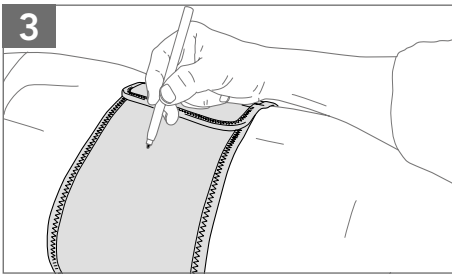
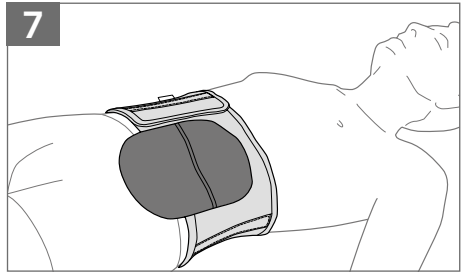
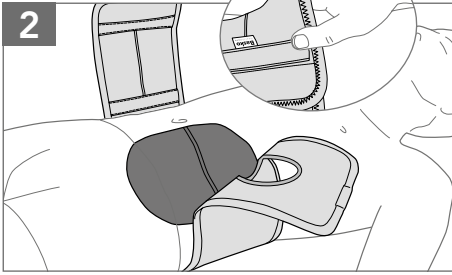
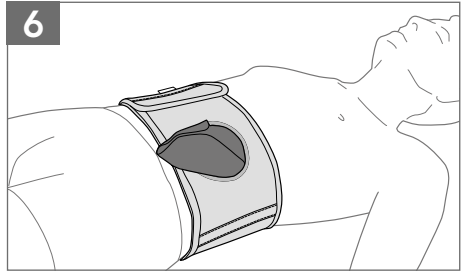
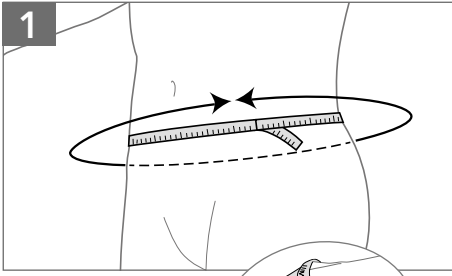
Basko

Healthcare


basko.com


Size chart Stomacare EasyCut Plus

Art. nr.	Size	Height	Abdominal Circumference	
S4100004-XXS	XXS	10 cm / 3.9"	XXS	50 - 60 cm / 19.7" - 23.6"
S4100004-XS	XS	10 cm / 3.9"		
S4100004-S	S	10 cm / 3.9"		
S4100004-M	M	10 cm / 3.9"		
S4100004-L	L	10 cm / 3.9"	XS	60 - 70 cm / 23.6" - 27.5"
S4100004-XL	XL	10 cm / 3.9"		
S4100004-XXL	2XL	10 cm / 3.9"		
S4100004-XXXL	3XL	10 cm / 3.9"	S	70 - 80 cm / 27.5" - 31.5"
S4100004-XXXXL	4XL	10 cm / 3.9"		
S4150004-S	S	15 cm / 5.9"	M	80 - 90 cm / 31.5" - 35.4"
S4150004-M	M	15 cm / 5.9"		
S4150004-L	L	15 cm / 5.9"		
S4150004-XL	XL	15 cm / 5.9"		
S4150004-XXL	2XL	15 cm / 5.9"	L	90 - 100 cm / 35.4" - 39.4"
S4150004-XXXL	3XL	15 cm / 5.9"		
S4150004-XXXXL	4XL	15 cm / 5.9"		
S4190004-S	S	19 cm / 7.5"	XL	100 - 110 cm / 39.4" - 42.9"
S4190004-M	M	19 cm / 7.5"		
S4190004-L	L	19 cm / 7.5"		
S4190004-XL	XL	19 cm / 7.5"		
S4190004-XXL	2XL	19 cm / 7.5"	XXL	110 - 120 cm / 42.9" - 47.2"
S4190004-XXXL	3XL	19 cm / 7.5"		
S4190004-XXXXL	4XL	19 cm / 7.5"		
S4230004-S	S	23 cm / 9.0"	XXXL	120 - 130 cm / 47.2" - 51.2"
S4230004-M	M	23 cm / 9.0"		
S4230004-L	L	23 cm / 9.0"		
S4230004-XL	XL	23 cm / 9.0"		
S4230004-XXL	2XL	23 cm / 9.0"	XXXXL	130 - 140 cm / 51.2" - 55.1"
S4230004-XXXL	3XL	23 cm / 9.0"		
S4230004-XXXXL	4XL	23 cm / 9.0"		

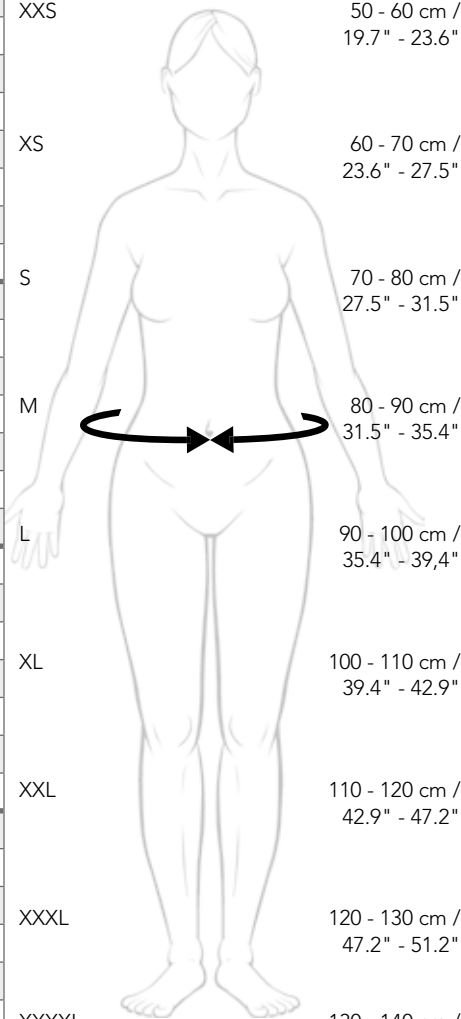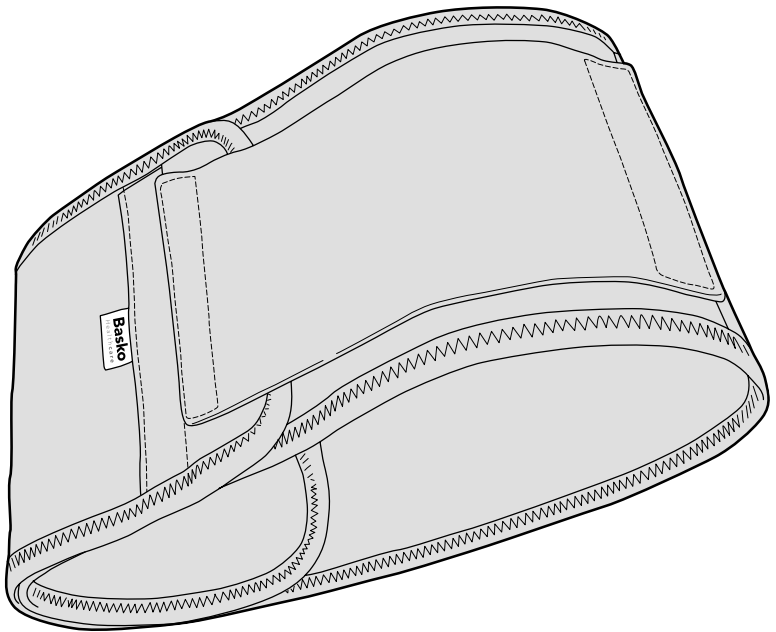NL - Gebruiksaanwijzing.....	2
DE - Gebrauchsanleitung	4
EN - Instructions for use.....	6
FR - Mode d'emploi	8
SE - Användarinstruktioner.....	10
NO - Bruksanvisning	12
DK - Brugervejledning	14
FI - Käyttöohje	16
IT - Manuale di istruzione	18
ES - Manual de instrucciones.....	20
PT - Manual de instruções	22
PL - Instrukcję obsługi.....	24

NL**DE****EN****FR****SE****NO****DK****FI****IT****ES****PT****PL**

EasyCut *Plus* Stomacare bandage

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EasyCut *Plus* Stomacare Ostomy Belt

Instructions for use

Your Stoma, wound nurse or physician has advised you to start wearing a stoma belt. For optimal care, the user should read these instructions carefully before applying the belt. This product is a medical device and is intended for use by one person only.

Intended use of the product

The EasyCut *Plus* belt is mainly used postoperatively to support the abdominal wall and prevent the formation of hernia and prolapse, for example during physical activities where pressure on the abdominal wall is higher (sports, gardening, lifting, vacuuming, etc.). The EasyCut *Plus* belt offers the wearer a high level of wearing comfort, a safe and secure feeling in daily life.

Packaging content

First check that the contents of the packaging are complete and undamaged. If this is not the case, please contact the person from whom you purchased the product directly. The packaging contains the following parts:

- Stomacare EasyCut *Plus* Ostomy Belt
- Aperture template
- Prolapse cover
- Instructions for use

Applying belt

To wear the EasyCut *Plus* belt correctly, you should preferably apply the belt while lying down. In case of a fracture, in many cases it will sink back into the abdomen and the belt provides counter pressure. This way the fracture is optimally supported. The belt is closed in such a way that you can still slide your hand in between. The belt is then sufficiently tight to provide support but still wears comfortably.

Fig. 1

The circumference of the abdomen must be measured to select the correct size. Use the size chart to determine which size is the best fit.

Fig. 2

Place the belt on the bed (or put it on loosely) and lie down on top of the belt. Close the front Velcro closure at the desired tension in such a way, a hand can still be inserted.

Fig. 3

Center the velour panel around the stoma-opening (or wherever desired) and mark this position with a marker. Now open the belt and remove it.

Fig. 4

Lay the belt flat and use the aperture template to mark the round or oval trimlines (depending on the pouch system) onto the belt.

Fig. 5

Use sharp scissors to cut an opening in the velour panel along the marked trimlines. The velour material does not fray.

Fig. 6

Attach the stoma pouch against the abdominal wall and over the stoma opening and lie down again. Fold the tip of the stoma pouch into a point and guide it from the inside of the belt out through the stoma aperture, with flange remaining at inside of the belt.

Fig. 7

Make sure the stoma pouch is nice flat and without folds and then position the belt over the abdomen and close it. After closing, check whether it's still possible to insert a hand between the belt and your abdomen. If necessary, adjust the tension. After completion you can stand up.

Fig. 8

Attach one end of the prolapse cover to the velour panel with the Velcro hook. Bring the prolapse cover to the desired tension and then secure the other end to the velour panel.

Fig. 9 (optional)

If chosen, first position the Protector (accessory that protects you from unwanted pressure on the stoma, e.g. due to seat belt) over the stoma pouch and press it tightly with the Velcro onto the belt. Then apply the prolapse cover as described.

Indication

- Abdominal wall weakness (ICD10: M79.9)
- Cicatricial and Stoma hernia (ICD10: K43.9)
- Colon Conduit (ICD10: Z93.9)
- Colostomy (ICD10: Z93.3)
- Enterostomy (ICD10: Z93.4)

Fig. 10 (optional)

You can also choose to apply a BellyBag (an accessory that helps to 'carry' the pouch, providing extra comfort and support) to the velour panel. Then place the prolapse cover as described.

- Ileam Conduit (ICD10: Z93.9)
- Ileostomy (ICD10: Z93.2)
- Prolapse (ICD10: K91.4)
- Ureterocutaneostoma (ICD10: Z93.6)
- Urostomy (ICD10: Z93.6)

Contraindications

There are no contraindications known.

Additional product information / area of use

The belt is not suitable for use in salt water.

Existing user risks

No user risks are known. However, the user must regularly check if pressure points or other skin irritations occur and must immediately contact the oEasyCut, wound nurse or technician if so.

Warning

The belt must not be worn too tight and must be tensioned in such a way that the hand can still be inserted. The bandage should not come into direct contact with open wounds and should be worn on dry skin. The use of lotion or creams is not recommended and should first be discussed with your stoma, wound nurse or physician.

Maintenance advice

If the belt is not worn, it is important to store it with the closures closed. To elongate the life of the belt, it is important that the Velcro straps remain free of dirt, dust etc. to continue to guarantee good adhesion.

Materials used

Polyester, Polyamide and Elastane.

Washing instructions

We recommend washing the belt in warm water at 40° C with a mild detergent. To prevent damage to the fabric, the Velcro closures must be closed prior to washing.

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